

HONEYWELL DC300

N95 Disposable Particulate Respirator

Comfortable protection so that you can work hard and breathe easy.

Honeywell DC300 series respirators meet NIOSH standards to provide 95% filtration efficiency against solid particulates and non-petroleum based liquid aerosols.

- **Comfortable fit** for a variety of face shapes and sizes, and is compatible with spectacles.
- **Hypo-allergenic** for users with sensitive skin.
- **Form-fitted mask** contours with the natural shape of the face, minimizing pressure points.
- **Filtration performance** provided by electrostatic non-woven polypropylene polyester fibers.
- **Dual welded head straps** keep the respirator comfortably on the face.
- **Latex-free** woven suspension straps reduce the possibility of breakage and do not pull or tangle hair.
- **Lightweight construction** allows for longer wear time and less fatigue.
- **Soft inner lining** for extended comfort.
- **Strong shell** prevents mask from collapsing in humid and moist environments.

HONEYWELL DC300 ORDERING INFORMATION



DC300 Series N95 Disposable Particulate Respirator

Part Number	DC300N95
Box Information	20 Masks / Box
UOM	Box

DC300 Series N95 Disposable Particulate Respirator with Exhalation Valve

Part Number	DC300N95V
Box Information	10 Masks / Box
UOM	Box

Honeywell

HONEYWELL DC300 DONNING



Step 1

Cup the respirator in the hand, allowing the head straps to hang freely



Step 2

Hold the respirator under the chin, extending lower head strap behind the neck and below the ears



Step 3

Extend the top head strap around the crown of the head and above the ears



Step 4

Place hands on each side of the respirator, adjusting position to achieve the optimal fit on the face

HONEYWELL DC300 USER INSTRUCTIONS

- Before use, the wearer must first be trained by the employer in the correct use of the respirator in accordance with applicable safety and health standards.
- OSHA standards, including 29 CFR 1910.134(f), and CSA standard Z94.4, require that the wearer be fit tested.
- Perform a seal check before using the respirator.
- Leave the contaminated area if breathing becomes difficult or if the respirator becomes damaged.
- Discard the respirator and replace with a new one if:
 - Excessive clogging of the respirator causes breathing difficulty.
 - The respirator becomes damaged.
 - The respirator becomes soiled, or contaminated with blood or bodily fluids.
- Do not use the respirator with beards or other facial hair that interfere with direct contact between the face and respirator sealing surface, or any other conditions that may prevent a proper seal



Warning

This document provides only an overview of the respiratory products shown. It does not provide all important product warnings and instructions. Honeywell recommends all users of respiratory equipment undergo thorough training and that all warnings and instructions provided with the products be thoroughly read and understood prior to use. Failure to read and follow all product warnings and instructions may result in serious personal injury, illness, or death.


Technical Proposal – Honeywell Solutions

The DC300N95 Series disposable respirator meets the National Institute for Occupational Safety and Health (NIOSH) standards for N95 filtration efficiency against solid particulates and non-petroleum based liquid aerosols.

Product Attributes:

- Exclusive design that contours with the natural shape of the face and withstands high humidity and moist environments.
- Moisture resistant filter media helps reduce the exhalation effort, thus improving breathing resistance.
- Universal size, designed to comfortably fit most facial shapes while providing outstanding respiratory protection. The form-fitted mask contours with the natural shape of the face, minimizing pressure points.
- Staple-free, dual-welded suspension straps provide a secure fit and remove the risk of skin irritation.
- The strong, rigid outer shell along with the channels reinforce the design structure, so the respirator holds its shape longer. The moisture-resistant soft inner lining feels smooth when contacted with the skin, extending comfort and ultimate wear time for the user.

DC300N95 Disposable

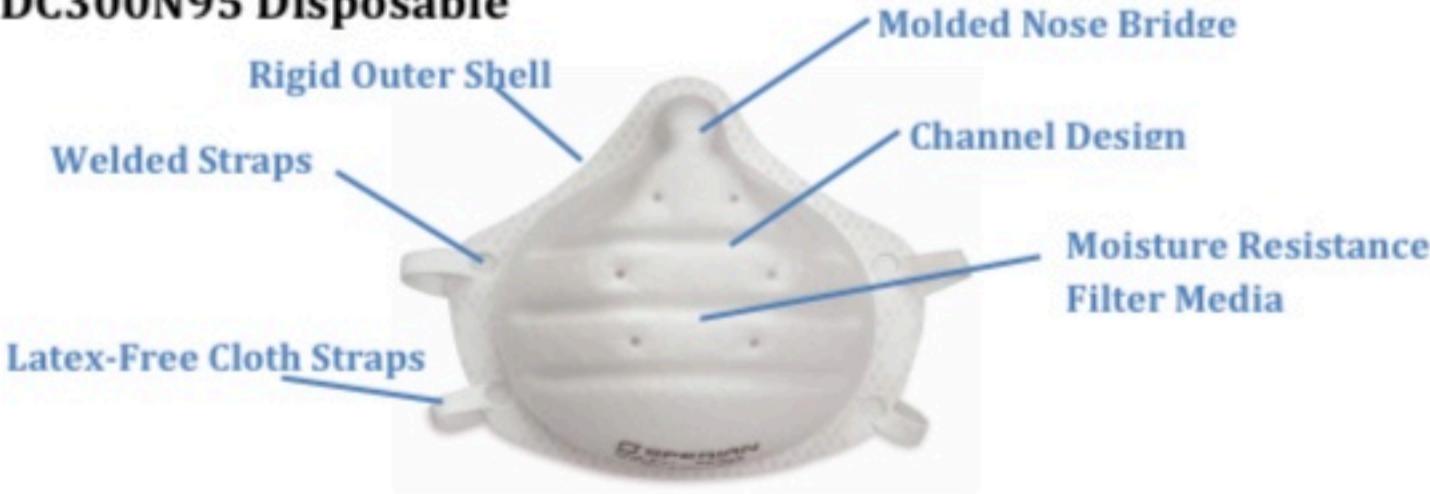

<p>Breathe Easier</p> <p>Multilayered absorption media, designed for a lower inhalation resistance makes it easier to inhale and pull air in.</p>	<p>Importance of Fit</p> <p>Contoured mask fits with the natural shape of the face, minimizing pressure points, providing optimal comfort and a secure fit.</p>	<p>Premium Filter Media</p> <p>Lower exhalation resistance, reduces heat buildup in the respirator by releasing the hot exhaled breath faster.</p>
<p>Lightweight</p> <p>Lightweight, thin construction with smooth inner lining for a comfortable fit.</p>	<p>Long Lasting</p> <p>Humidity and moisture resistant filter media that repels moisture for a longer lasting respirator.</p>	<p>Trusted</p> <p>95% filtration efficiency against solid particulates and non-petroleum based liquid aerosols.</p>